

Multi-Drug Urine Rapid Test - Diazepam, Alprazolam & Flunitrazepam

- 3 in 1 Urine Rapeid Drug Test
- Detects Valium, Xanax & Rohypnol
- One step, easy to use, self-testing device
- Accurate with results in 5 minutes

WHAT IT TESTS FOR AND THE LEVELS

- Diazepam (Valium) - 300 ng/mL
- Flunitrazepam (Rohypnol) - 500 ng/mL
- Alprazolam (Xanax) - 100 ng/mL


Please Note, this test is for URINE only, it won't work on powders

DIRECTIONS FOR USE

Allow the test, urine specimen, and/or controls to reach room temperature (15-30°C) prior to testing.

1. Bring the pouch to room temperature before opening it. Remove the test panel from the sealed pouch and use it as soon as possible.
2. Remove the cap.
3. With the arrow pointing toward the urine specimen, immerse the test panel vertically in the urine specimen for 10 to 15 seconds. **Immerse the dipstick to at least the level of the wavy lines, but not above the arrow on the test panel.**
4. Replace the cap and place the test panel on a non-absorbent flat surface.
5. Start the timer and wait for the colored line(s) to appear.
6. **Read the adulteration strips and alcohol strip between 3-5 minutes** according to the color chart provided separately /on foil pouch. Refer to your Drug Free Policy for guidelines on adulterated specimens. We recommend not to interpret the drug test results and either retest the urine or collect another specimen in case of any positive result for any adulteration test.
7. The drug strip results should be read at **5 minutes**. Do not interpret the result after 10 minutes.

INTERPRETATION OF RESULTS

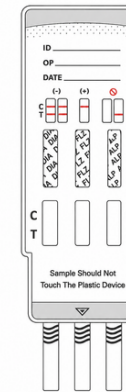

NEGATIVE: A coloured line appears in the control region (C) and another coloured line appears in the test region (T). This negative result means that the concentrations in the urine sample are below the designated cut-off levels for a particular drug tested.

***NTOE:** The shade of the coloured line(s) in the test region (T) may vary. The result should be considered negative whenever there is even a faint line.



POSITIVE: A coloured line appears in the control region (C) and no line appears in the test region (T). The positive result means that the drug concentration in the urine sample is greater than the designated cut-off for a specific drug.



INVALID: No line appears in the control region (C). Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for control line failure. Read the directions again and repeat the test with a new test. If the result is still invalid, contact your manufacturer.